

CEREC Primescan

1. MANUFACTURER: Dentsply Sirona
2. USE:
 - ☐ FULL ARCH SCAN
 - ☐ DOES **NOT** REQUIRE POWDER FOR INTRAORAL SCANNING
3. LOCATION: S434



<https://www.dentsplysirona.com/en-us/discover/discover-by-topic/by-category/digital-impression/digital-impression-for-dental-practices.html>

<https://www.dentsplysirona.com/content/dam/master/product-procedure-brand-categories/digital-impression/product-categories/intraoral-scanners/documents/DIM-flyer-Primescan-Implants-workflow-A91100-M43-C313-EN.pdf>

CEREC Primescan


Digital Impression with Primescan

Outstanding quality	Security	Predictability
<ul style="list-style-type: none"> Highly accurate full scan-arch in less than a minute Easy scanning of shiny materials and edentulous areas Scan as the foundation for better function, and precision of the prosthetic solutions in implant treatment planning 	<ul style="list-style-type: none"> Guided implant surgery with surgical guides for a more effective implant workflow Increased treatment comfort for your patients Expect more predictable and stable placements with DS PrimeTaper* 	<ul style="list-style-type: none"> Reliable data for extremely predictable implant treatment Complete control even over more complex cases 95% of Primescan clinicians are likely to recommend Primescan to colleagues*



<https://www.dentsplysirona.com/en-us/discover/discover-by-topic/by-category/digital-impression/digital-impression-for-dental-practices.html>

<https://www.dentsplysirona.com/content/dam/master/product-procedure-brand-categories/digital-impression/product-categories/intraoral-scanners/documents/DIM-flyer-Primescan-Implants-workflow-A91100-M43-C313-EN.pdf>

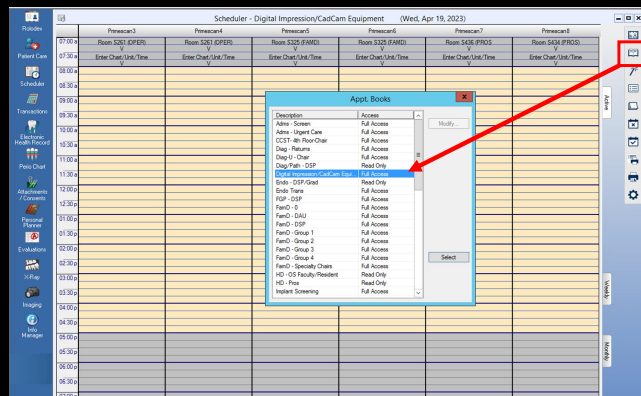

- Please reserve the scanner before appointment **with the front desk.**
- Please make sure your name and clinic unit are in the schedule when booking the equipment!
- All equipment **must be returned by end of a morning or afternoon session** in order to meet infection control protocol in preparation for use on the following day.

Mirror sleeve processing protocol


For patient cases (Primescan)



- Go to the dispensary on **4th floor** to check out a mirror sleeve for the Primescan. When the mirror sleeve is in a paper plastic pouch with a clean and disinfected by label on the pouch the mirror sleeve is ready to be placed on the camera body and used on a patient.
- After the procedure is completed, remove the mirror sleeve from the camera body (**do not twist**) and place the **white sleeve** to protect the camera **before** cleaning.
- Deliver the sleeve back to the **same** dispensary in a plastic pouch. **Sleeve must be handed to a dispensary staff.**
- Sleeve not checked back in by staff will run the chance of it getting stolen and any damage to the sleeve left in dispensary window.
- Please put the white sleeve back in place and return the scanner to the storage room.

All equipment must be returned by end of a morning or afternoon session in order to meet infection control protocol in preparation for use on the following day.


CEREC Primescan

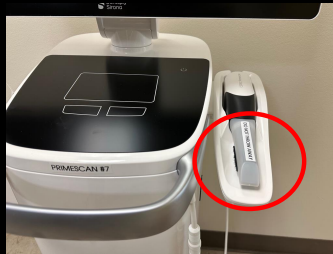

Check out a mirror sleeve
for the Primescan at the
dispensary




CEREC Primescan

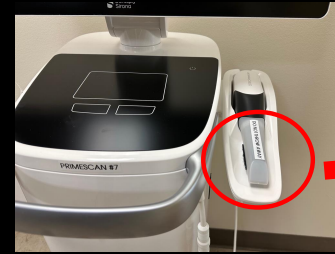

PLEASE DO NOT THROW THE WHITE SLEEVE AWAY!

CEREC Primescan

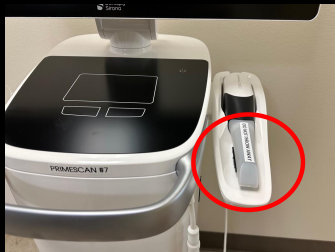

After the procedure is completed,
remove the mirror sleeve from the
camera body (**do not twist**) and place
the white sleeve to protect the
camera **before** cleaning.
(Use CaviWipes for cleaning)

CLEAN THE SCANNER WITH CaviWipes ONLY!!, NOT OPTIM 1 Wipes



Mirror sleeve processing protocol

For patient cases (Primescan)



Deliver the sleeve back to the **same** dispensary in a plastic pouch.

Mirror sleeve processing protocol

For patient cases (Primescan)



DO NOT PUT THE SLEEVE AT THE DISPENSARY WINDOW!



Sleeve must be handed to a dispensary staff.



Ensure that the cord is properly hung and organized at the back of the machine before returning the scanner to the storage room (S434).